Affordable. Reliable. Home Improvement.

OVERBED TABLES

MODEL: GT-SR65-B7

Installation Guide:

https://www.youtube.com/watch?v=_nftsqNhAeU&list=PL0BwQM2YkbL6_nMedm_7wc2tVTtxGLA3f&index=2

Technical Support and E-Warranty Certificate


www.vevor.com/support

Affordable. Reliable. Home Improvement.

OVERBED TABLES

MODEL: GT-SR65-B7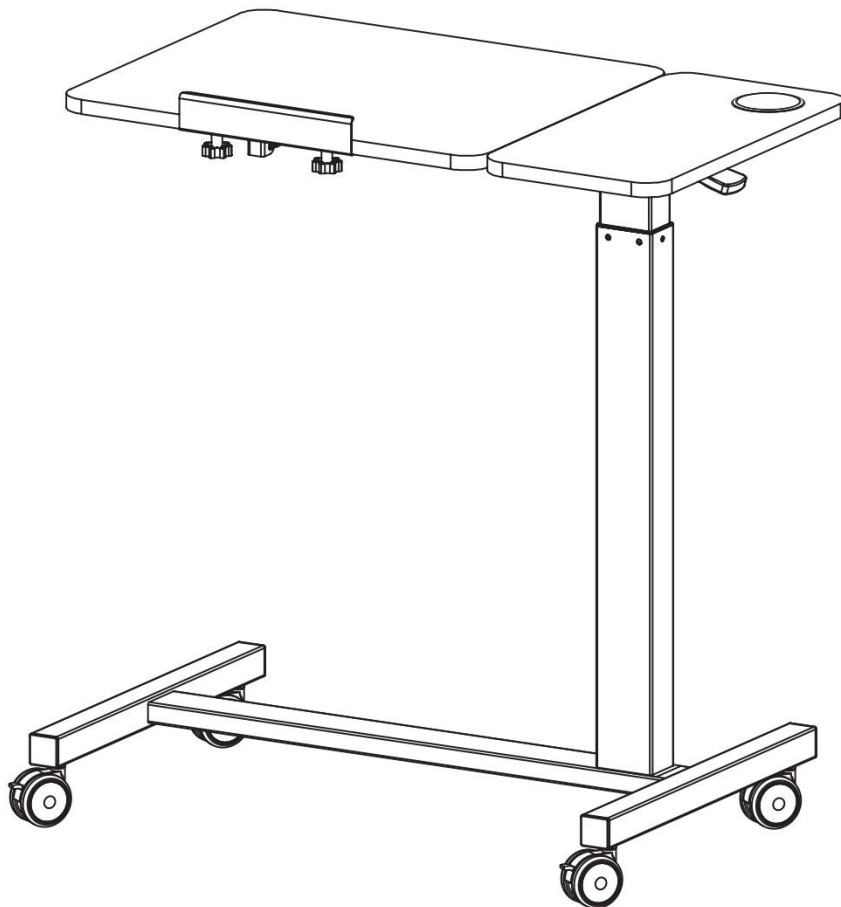

This is the original instruction, please read all manual instructions carefully before operating. VEVOR reserves a clear interpretation of our user manual. The appearance of the product shall be subject to the product you received. Please forgive us that we won't inform you again if there are any technology or software updates on our product.

SAFETY INSTRUCTIONS

WARNING:

Read this material before using this product. Failure to do so may result in serious injury.

Assembly precautions


1. Assemble only according to these instructions. Improper assembly may create hazards.
2. Wear ANSI-approved safety goggles and heavy-duty work gloves during assembly.
3. Keep the assembly area clean and well-lit.
4. Keep bystanders out of the area during assembly.
5. Do not assemble if tired or when under the influence of alcohol, drugs or medication.
6. The product capabilities apply to properly and completely assembled products only.
7. Assemble on a flat, level, hard and smooth surface capable of safely supporting the Overbed Tables.
8. For additional information regarding the parts listed in the following pages, please refer to the Assembly Diagram of this manual. Unwrap and separate all parts in a clean work area.

Use precautions

1. DO NOT SIT OR STAND ON THIS ITEM.
2. This product is not a toy. Do not allow children to play with or near this item.
3. Do not exceed specified weight capacities.
4. Use only on a flat, level, hard, and smooth surface that can safely support a fully loaded Overbed Tables.
5. Use as intended only.
6. Inspect before every use; do not use if parts are loose or damaged.

SAVE THIS MANUAL


PARTS LIST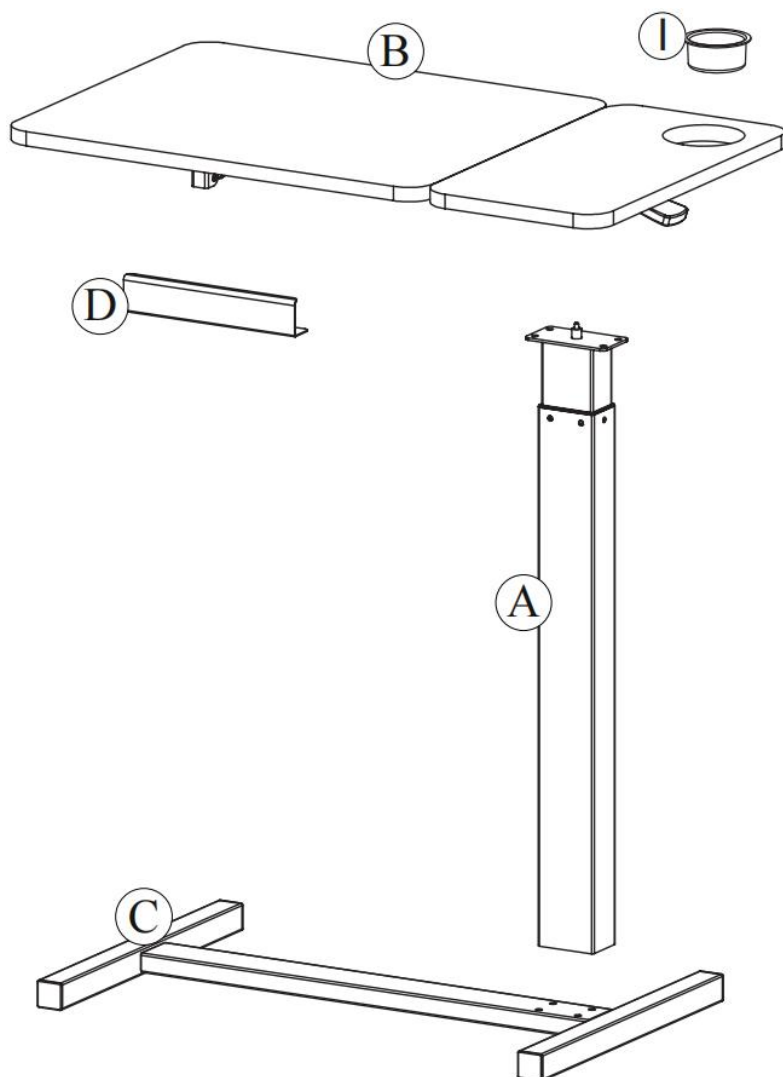

	Ex4	M6x12mm		Fx4	M6x25mm
	Gx2			Hx1	4mm
	Kx1	M14			
	Lx2			Mx2	


ASSEMBLY STEP


STEP1: Installation of lifting bar.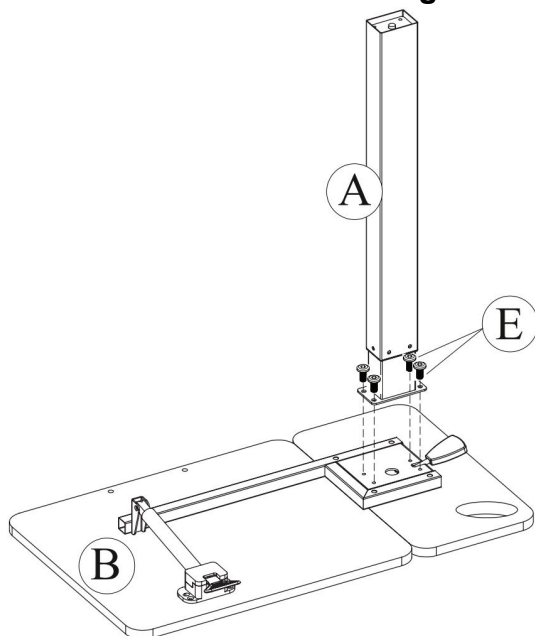


	
E x 4	H x 1

STEP2: Mounting feet.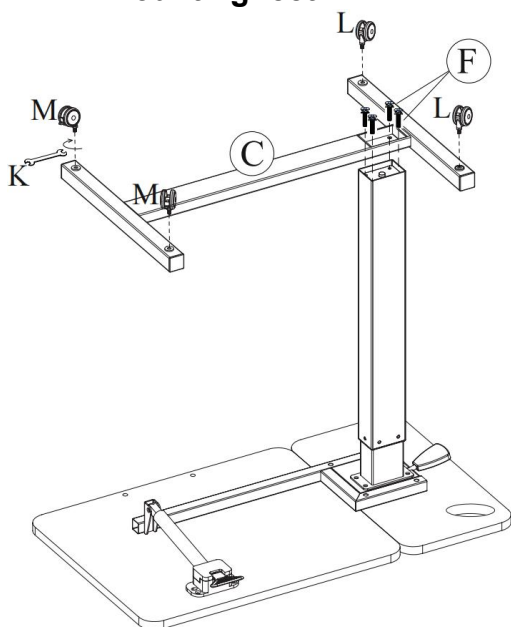


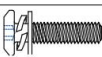	
	K x 1
	
L x 2	M x 2
	
F x 4	H x 1


STEP3: Placement of cup holders.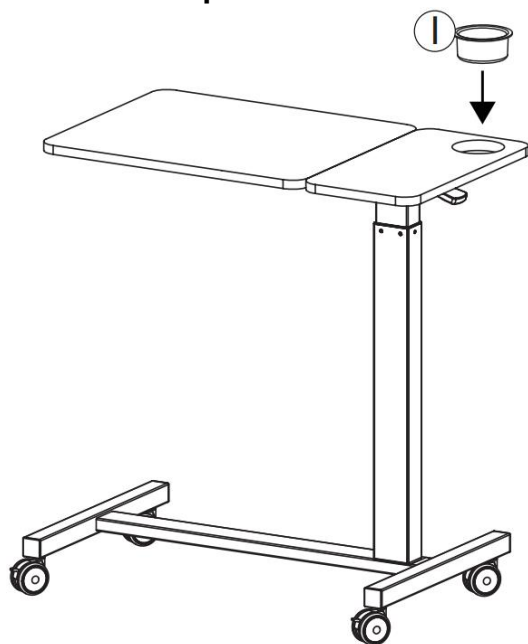


STEP4: Installation of baffle.

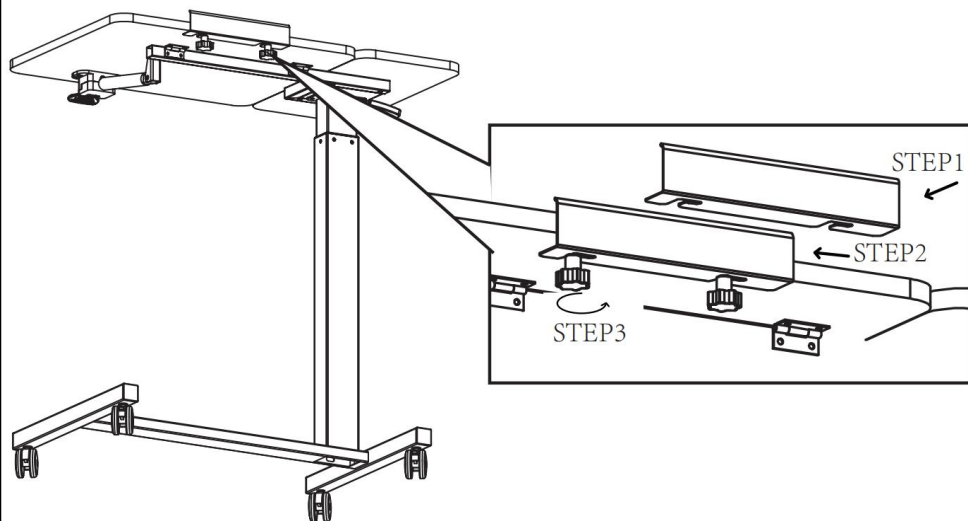Desktop lifting operation mode:

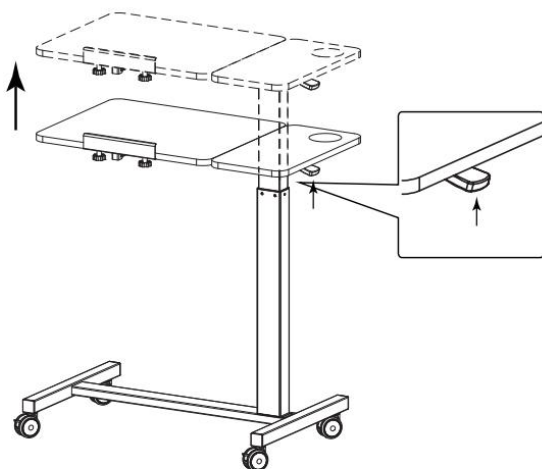

Lift the paddle for lifting the table top.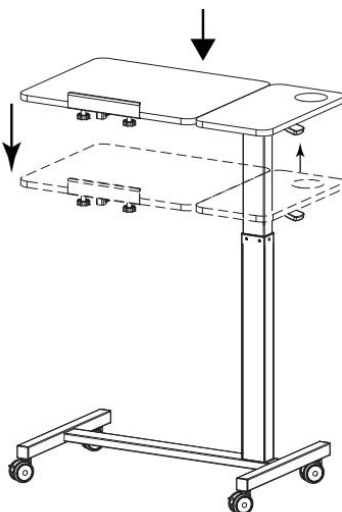

Lift the paddle and press the table top for downward operation.

Desktop flipping operation mode:

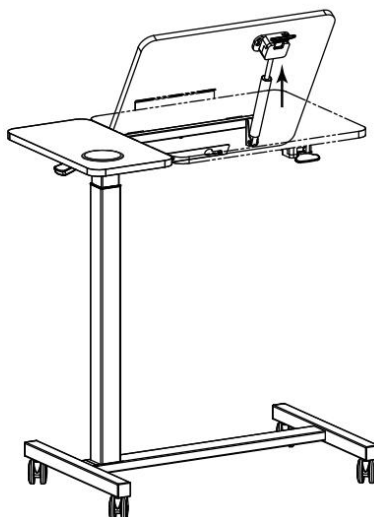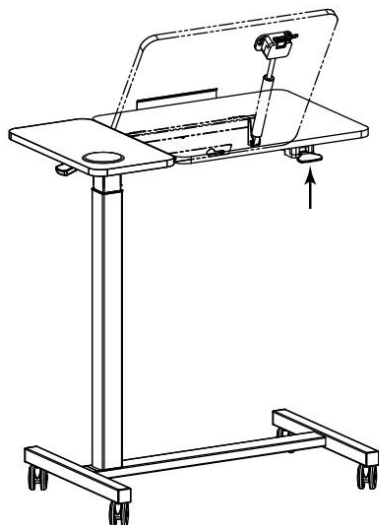Turn the crank to rotate the surface to 60°.

WARNING: There is a risk that objects on the surface may fall off when rotating the surface. Before performing this movement, make sure that there are no objects on the surface that could fall off and be damaged.

PRODUCT PARAMETER

Model	GT-SR65-B7
Safe Loading Weight	30kg near columns 10kg at table edge
Colour	Wood + Black
Angle of Rotation	0~60°
Product Size	700*400*1085mm (highest position) 700*400*705mm (lowest position)

Manufacturer: Shanghaimuxinmuyeyouxiangongsi

Address: Shuangchenglu 803nong11hao1602A-1609shi, baoshanqu, shanghai 200000 CN.

Imported to AUS: SIHAO PTY LTD. 1 ROKEVA STREETEASTWOOD NSW 2122 Australia

Imported to USA: Sanven Technology Ltd. Suite 250, 9166 Anaheim Place, Rancho Cucamonga, CA 91730

YH CONSULTING LIMITED. C/O YH Consulting
Limited Office 147, Centurion House, London
Road, Staines-upon-Thames, Surrey, TW18 4AX

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Importerad till AUS: SIHAO PTY LTD. 1 ROKEVA STREETEASTWOOD NSW 2122
Importerad till USA: Sanven Technology Ltd. Suite 250, 9166 Anaheim Place,
E-CrossStu GmbH