VEVOR[®]

TOUGH TOOLS, HALF PRICE

Technical Support and E-Warranty Certificate www.vevor.com/support

HUMAN BODY TEACHING MODELS USER MANUAL

MODEL:LD-140

We continue to be committed to provide you tools with competitive price.


"Save Half", "Half Price" or any other similar expressions used by us only represents an estimate of savings you might benefit from buying certain tools with us compared to the major top brands and does not necessarily mean to cover all categories of tools offered by us. You are kindly reminded to verify carefully when you are placing an order with us if you are actually saving half in comparison with the top major brands.

VEVOR®

TOUGH TOOLS, HALF PRICE

HUMAN BODY TEACHING
MODELS

MODEL:LD-140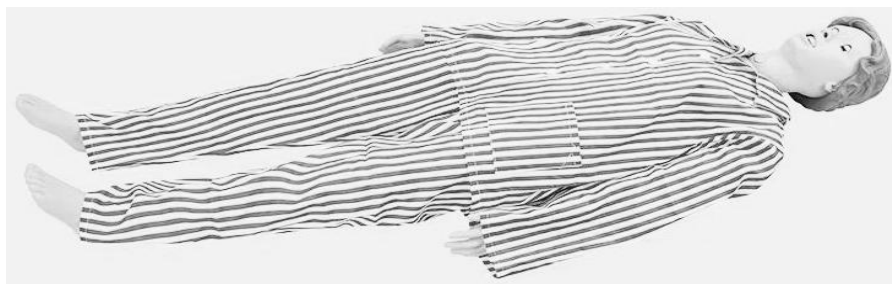

NEED HELP? CONTACT US!

Have product questions? Need technical support? Please feel free to contact us:

Technical Support and E-Warranty Certificate
www.vevor.com/support

This is the original instruction, please read all manual instructions carefully before operating. VEVOR reserves a clear interpretation of our user manual. The appearance of the product shall be subject to the product you received. Please forgive us that we won't inform you again if there are any technology or software updates on our product.

SAFETY INSTRUCTIONS

WARNING:

1. Read these instructions before using this product. Failure to do so can result in serious injury.
2. After use, please assemble all the parts and keep them vertically.
3. Avoid high temperature and direct sunlight.
4. Remove minor stains with neutral detergent. The use of alcohol, benzene and other substances may damage the product.

SAVE THESE INSTRUCTIONS

INSTRUCTIONS

Model features: Male and female

This model is designed according to the requirements of clinical basic nursing operation and practice nursing technical outline. Using imported thermoplastic elastic-plastic materials, with integral nursing and disassembly parts for skill training teaching, is an ideal teaching practice model for clinical medical staff. It is characterized by vivid image, real operation, convenient disassembly, reasonable structure, durable, and parts can be replaced, is the most advanced, the most complete function, materials elegant new products.

Product functions

- 1, the simulation of desirable supine kneeling position, two legs after abduction can be independent support, left and right upper arm, calf can be flexible rotation
- 2, bed sore care: show pressure sore clinical stage 4 different stages, the first stage: congestion and red; The second stage: inflammatory infiltration stage; Stage 3: shallow ulcer stage; Stage 4: necrotic ulcer stage. At the same time, pressure sore and various pathological manifestations: pressure sore inflammation, ulcers, sinuses, carrion, necrosis, eschar, etc 3, head

- care:shampoo,face wash,eye and ear drops cleaning,oral care
- 4.holistic care:bed bath,sitting bath,wear and change clothes,cold and heat therapy
- 5.can conduct oral direct tracheal intubation operation training
- 6.Sputum aspiration method
- 7.Oxygen inhalation method
- 8.Oral and nasal feeding method
- 9.Gastric lavage
- 10.Structure observation of vital organs in thoracic anatomy
- 11.Observation of the structure of vital organs in abdominal anatomy
- 12.Arm venipuncture,injection and infusion (blood)
- 13.Subcutaneous injection into deltoid muscle
- 14.Injection into lateral femoris muscle
- 15.Thoracic cavity and abdominal cavity puncture
- 16.Enema method
- 17.Female catheterization
- 18.Male catheterization
- 19.Female bladder irrigation
- 20.Male bladder irrigation
- 21.Fistula drainage
- 22.Hip intramuscular injection
- 23.Limb joints bend left and right,rotate and move up and down

Scope of Application

- 1.Clinical teaching and practical operation training for students in medical universities,nursing schools and medical colleges
- 2.Continuing clinical teaching practice training for hospital medical and nursing staff
- 3.Popularize clinical medicine training in grassroots health units

Maintenance,warranty

- 1.After the training of nasal feeding,gastric lavage,enema,catheterization and other functional operations,the residual liquid injected into the

stomach,intestine and

bladder should be emptied during the operation.

2.Keep the model clean.After the operation training,the model should be scrubbed and cleaned with clean water cloth,and chemical solvents should be prohibited.

3.If not used for a long time,please clean up the model and store it in a ventilated,cool and dry place to prolong the service life.

4.Within one year of the purchase date of this model or the warranty period stipulated in the contract,if any quality problems occur due to the failure of the


manufacturing process or the original (except man-made),the company is responsible for free maintenance services,and promises lifetime maintenance (the cost is at the user's own expense).

PRODUCT PARAMETER

Model	LD-140
Gender	Woman/Man
Material	PVC

PARTS LIST

Clothes	1	
Syringe 10ml	1	
Urinary Catheter	1	
Sputum Suction Tube	1	

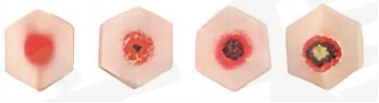Anal Canal	1	
Gastric Tube	1	
Bottle	3	
Infusion Bottle	1	
Wig	1	
Genital Organ	1	
Bedsore Module	4	

Manufacturer: Shanghaimuxinmuyeyouxiangongsi
Address: Shuangchenglu 803nong11hao1602A-1609shi,
baoshanqu, shanghai 200000 CN.
Imported to AUS: SIHAO PTY LTD. 1 ROKEVA STREET-
EASTWOOD NSW 2122 Australia
Imported to USA: Sanven Technology Ltd. Suite 250, 9166
Anaheim Place, Rancho Cucamonga, CA 91730

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YH CONSULTING LIMITED.
C/O YH Consulting Limited Office 147, Centurion House,
London Road, Staines-upon-Thames, Surrey, TW18 4AX

VEVOR[®]

TOUGH TOOLS, HALF PRICE

Technical Support and E-Warranty Certificate

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E-CrossStu GmbH