VEVOR[®]

TOUGH TOOLS, HALF PRICE

Technical Support and E-Warranty Certificate www.vevor.com/support

HUMAN BODY TEACHING MODELS

USER MANUAL

MODEL:LD-J140

We continue to be committed to provide you tools with competitive price.

"Save Half", "Half Price" or any other similar expressions used by us only represents an estimate of savings you might benefit from buying certain tools with us compared to the major top brands and does not necessarily mean to cover all categories of tools offered by us. You are kindly reminded to verify carefully when you are placing an order with us if you are actually saving half in comparison with the top major brands.

VEVOR[®]

TOUGH TOOLS, HALF PRICE

HUMAN BODY TEACHING
MODELS

MODEL:LD-J140

NEED HELP? CONTACT US!

Have product questions? Need technical support? Please feel free to contact us:

Technical Support and E-Warranty Certificate
www.vevor.com/support

This is the original instruction, please read all manual instructions carefully before operating. VEVOR reserves a clear interpretation of our user manual. The appearance of the product shall be subject to the product you received. Please forgive us that we won't inform you again if there are any technology or software updates on our product.

SAFETY INSTRUCTIONS

WARNING:

1. Read these instructions before using this product. Failure to do so can result in serious injury.
2. After use, please assemble all the parts and keep them vertically.
3. Avoid high temperature and direct sunlight.
4. Remove minor stains with neutral detergent. The use of alcohol, benzene and other substances may damage the product.

SAVE THESE INSTRUCTIONS

INSTRUCTIONS

Baby intubation model commonly known as infant airway obstruction and CPR model, was designed according to the principle of infant airway foreign body intubation. It has the characteristics of perfect shape, advanced function, careful material and real operation.

The skin of the head, hands and feet skin imported thermoplastic elastomer mixed plastic materials, stainless steel mold, injection molding machine high temperature injection pressure from the head, neck shell, chin, and chest pressure mechanism by imported ABS material, injection molded. The trunk from cotton, and polyester fabrics, with an accurate flag, feels real, realistic shape, beautiful appearance, and durability.

The product can be carried out the sea in children's first aid operation. Under pediatric anatomical features and physiological characteristics of the analog design, suitable for tracheal foreign bodies in an emergency. Tracheal foreign body is common in infants and young children, immature infant epiglottis function is not perfect, mouth and cracked talking, laughing and crying, and vigorous activity, it is easy to mouth with aspiration of endotracheal lead to asphyxiation caused by airway obstruction.

Features:

1. normal airway obstruction analog to;
 2. standard CPR operation: artificial respiration and cardiac compressions;
 3. chest expansion when the airway used to pass;
 4. choking, foreign body airway obstruction analog;
- Real scale of the standard infant and the accurate standard layout;
6. the precise anatomic structure, palpable sternum and ribs.

First Aid:

1. Back row hit: children straddle and prone on the first aid arm, the head lower than the trunk, fixed head and arm on the thigh of first aid, and then with the other hand grasp of force row hit two children in the back between the shoulder blades 4-6 times, a sudden increase in the respiratory tract internal pressure, helps to loosen the foreign body and excreted.
2. Chest finger slam method: In the child's supine position, hold the first aid arm bend the head slightly lower than the trunk, first aid with two fingers pressed between the nipples and the sternum midline junction point a transverse finger 4 - 6. If necessary, the above methods are used interchangeably. Until expelled or children with loss of consciousness.

How to use:

First check Baby infarction model is a choking.

Inspection method: Take a piece of disposable PVC the disinfection barrier mask shop in an analog baby face, do mouth to the nose and mouth artificial respiration, using his head and lift the chin method to open the airway, not thrown back over the head. Artificial respiration, if the analog chest of children and downs, there is no choking, to take chest the thumb pressing resuscitation processing. If the analog chest of children can not be ups and downs, for the simulation of sick infants' airway choking, take the following treatment procedures:

1. The first of the infarct foreign body (black sponge) into the mouth throat, analog the infant airway infarction site.
2. Then, the analog baby straddles the side of the forearm, palm behind the

neck, to mandibular angle fixed analog baby with the other hand, make the simulation of the baby's head slightly thrown back, open the airway.

3. Both hands forearm baby, flip was prone position; root percussion analog baby back between the shoulder with the palm of your hand four times, check the mouth, such as foreign body (black sponge) spit up, to take the hand of foreign body treatment.

4. In another way, his hands forearm simulated infant airway obstruction and CPR model is fixed, flipped to the supine position, fast impulsive baby nipples pressing the analog the next thumb at 4 times, dental check-up, such as foreign body (black sponge) spit up, quickly take hand foreign body method processing.

5. Such as obstruction can not spit, repeat back percussion and chest impact action several times.

PRODUCT PARAMETER

Model	LD-140
Gender	Baby
Material	PVC

PARTS LIST

Rib Simulator	1	
----------------------	---	---

Lung Simulation Package	4	
The Foreign Body Mimics a Sponge	24	
CPR Barrier Antiseptic Mask	10	
Storage Bag	1	

Manufacturer: Shanghaimuxinmuyeyouxiangongsi
Address: Shuangchenglu 803nong11hao1602A-1609shi,
baoshanqu, shanghai 200000 CN.
Imported to AUS: SIHAO PTY LTD. 1 ROKEVA STREET-
EASTWOOD NSW 2122 Australia
Imported to USA: Sanven Technology Ltd. Suite 250, 9166
Anaheim Place, Rancho Cucamonga, CA 91730

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YH CONSULTING LIMITED.
C/O YH Consulting Limited Office 147, Centurion House,
London Road, Staines-upon-Thames, Surrey, TW18 4AX

VEVOR[®]

TOUGH TOOLS, HALF PRICE

Technical Support and E-Warranty Certificate

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E-CrossStu GmbH