VEVOR[®]

TOUGH TOOLS, HALF PRICE

Technical Support and E-Warranty Certificate www.vevor.com/support

LIFT RECLINER CHAIR

USER MANUAL

MODEL NO.: SX-81541-2

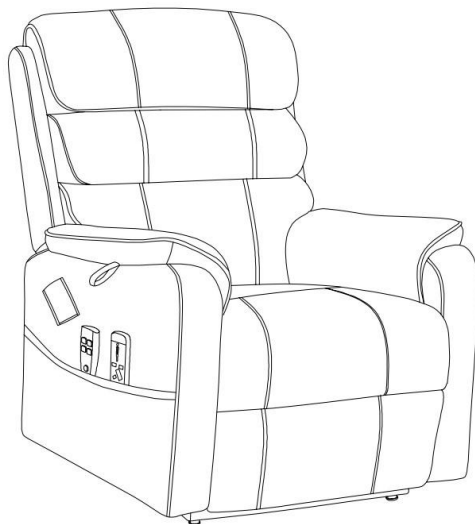
We continue to be committed to provide you tools with competitive price. "Save Half", "Half Price" or any other similar expressions used by us only represents an estimate of savings you might benefit from buying certain tools with us compared to the major top brands and does not necessarily mean to cover all categories of tools offered by us. You are kindly reminded to verify carefully when you are placing an order with us if you are actually Saving Half in comparison with the top major brands.

VEVOR[®]

TOUGH TOOLS, HALF PRICE

Lift Recliner Chair

MODEL NO.: SX-81541-2

NEED HELP? CONTACT US!

Have product questions? Need technical support? Please feel free to contact us:

Technical Support and E-Warranty Certificate
www.vevor.com/support

This is the original instruction, please read all manual instructions carefully before operating. VEVOR reserves a clear interpretation of our user manual. The appearance of the product shall be subject to the product you received. Please forgive us that we won't inform you again if there are any technology or software updates on our product.

	Warning-To reduce the risk of injury, user must read instructions manual carefully.
	Compliance is a CE security certification.
	This product is subject to the provision of European Directive 2012/19/EC. The symbol showing a wheeled bin crossed through indicates that the product requires separate refuse collection in the European Union. This applies to the product and all accessories marked with this symbol. Products marked as such may not be discarded with normal domestic waste, but must be taken to a collection point for recycling electrical and electronic devices

ASSEMBLY PRECAUTIONS

1. Assemble only according to these instructions. Improper assembly may create hazards.
2. Wear ANSI-approved safety goggles and heavy-duty work gloves during assembly.
3. Keep the assembly area clean and well-lit.
4. Keep bystanders out of the area during assembly.
5. Do not assemble when tired or when under the influence of alcohol, drugs or medication.
6. The product capabilities apply to properly and completely assembled products only.
7. Assemble on a flat, level, hard and smooth surface capable of safely supporting the Lift Recliner Chair.

8. For additional information regarding the parts listed in the following pages, please refer to the Assembly Diagram of this manual. Unwrap and separate all parts in a clean work area.

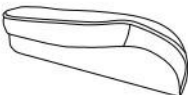
SAVE THESE INSTRUCTIONS

USE PRECAUTIONS

TO PREVENT SERIOUS INJURY AND DEATH FROM TIPPING:

1. This product is not a toy. Do not allow children to play with or near this item.
2. Do not exceed weight capacities. Empty Lift Recliner Chair before moving.
3. Use only on a flat, level, hard and smooth surface capable of safely supporting a fully loaded Lift Recliner Chair.
4. Use as intended only. Inspect before every use; do not use if parts are loose or damaged.

PACKAGE CONTENTS

Item no.	Reference Image	Qty.	Item no.	Reference Image	Qty.
A		X1	G		X4
B		X1	H		X1
C		X1	I		X1
D		x2	J		X1
E		x4	K		X1
F		X4	L		X1

ASSEMBLY STEPS

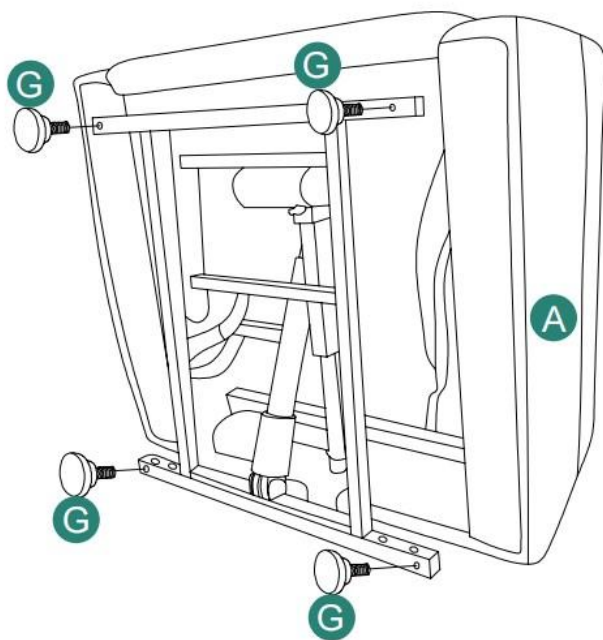
1

ⓐ 4pc

ⓐ 1pc

2

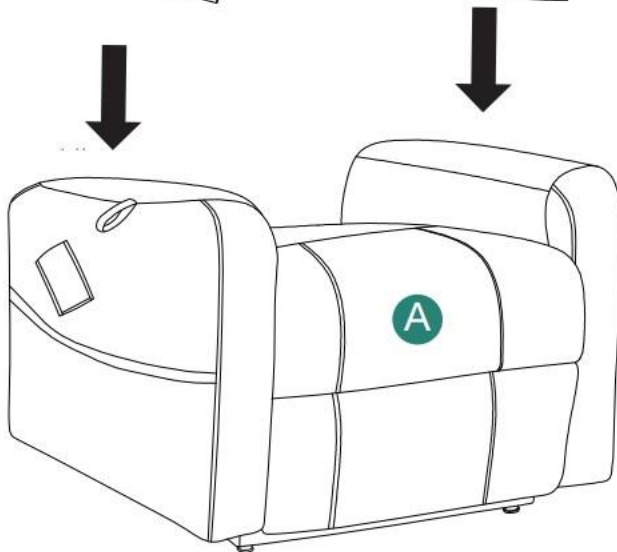
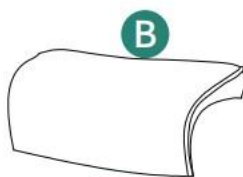
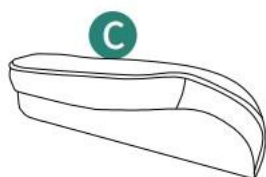
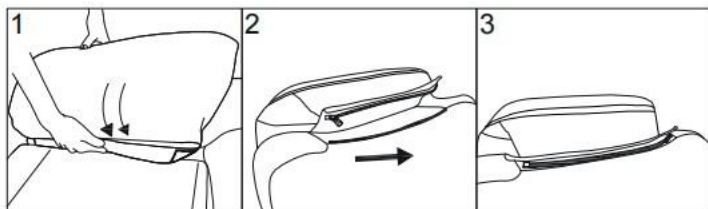
Ⓑ 1pc

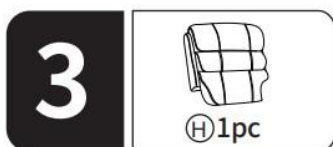
Ⓒ 1pc

Ⓐ 1pc

4

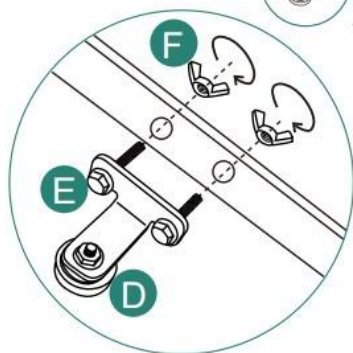
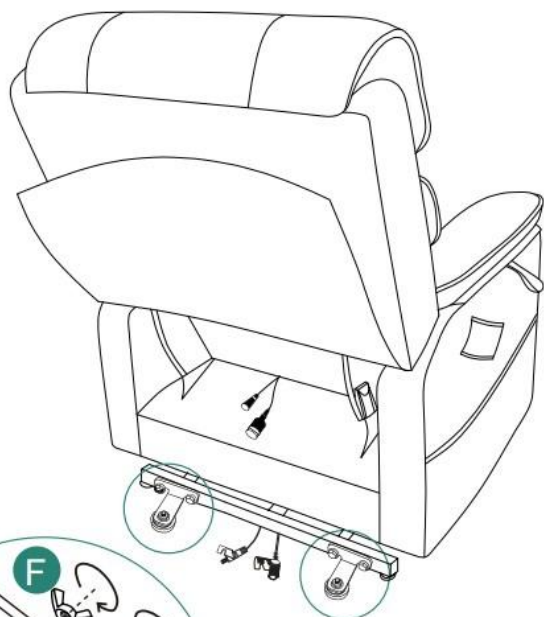
Ⓓ2pc

Ⓔ4pc

Ⓕ4pc

5

① 1pc

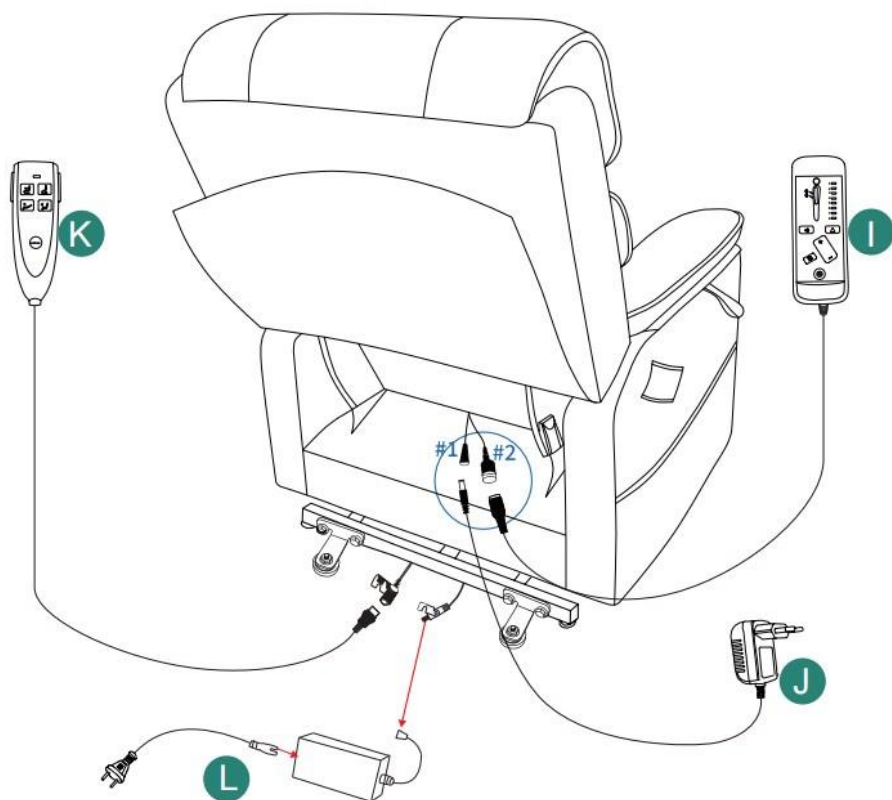
② 1pc

③ 1pc


④ 1pc


Connect wire #1 to part J, wire #2 to part I.

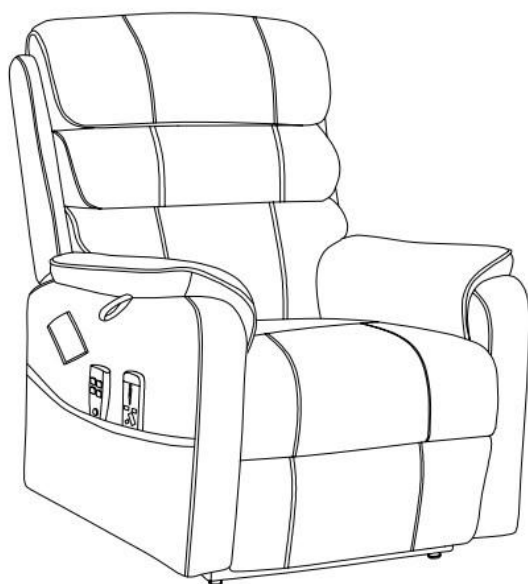6


How to use the massage function?

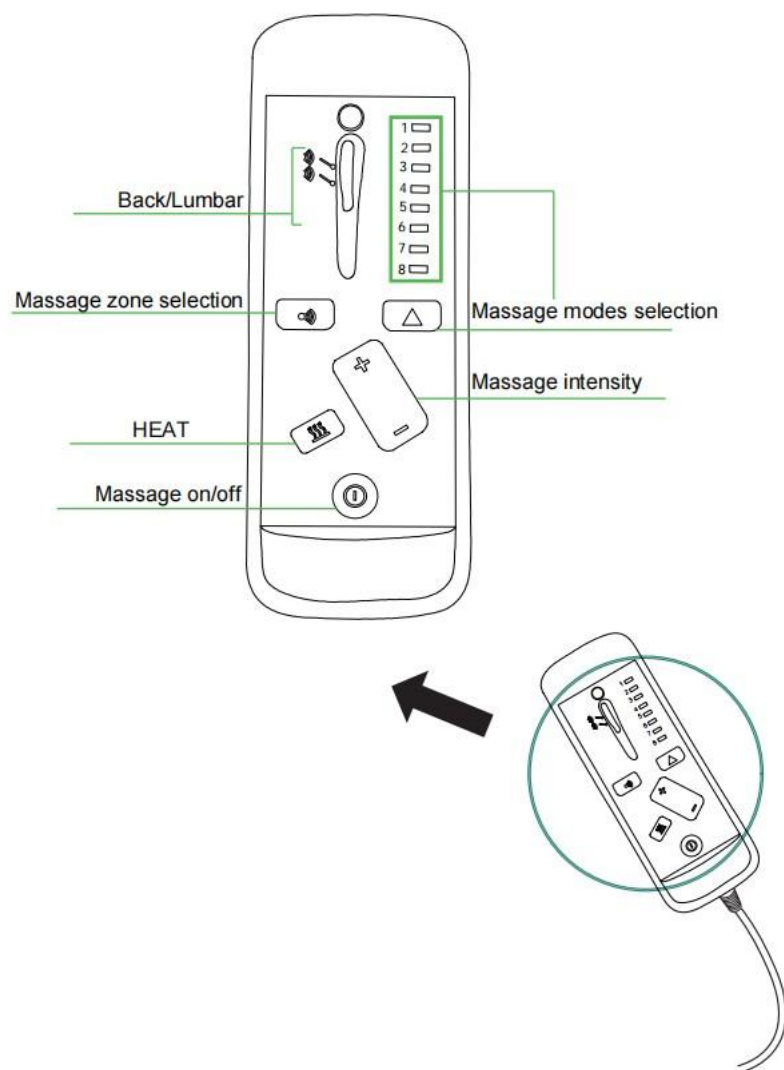INFINITE POSITION

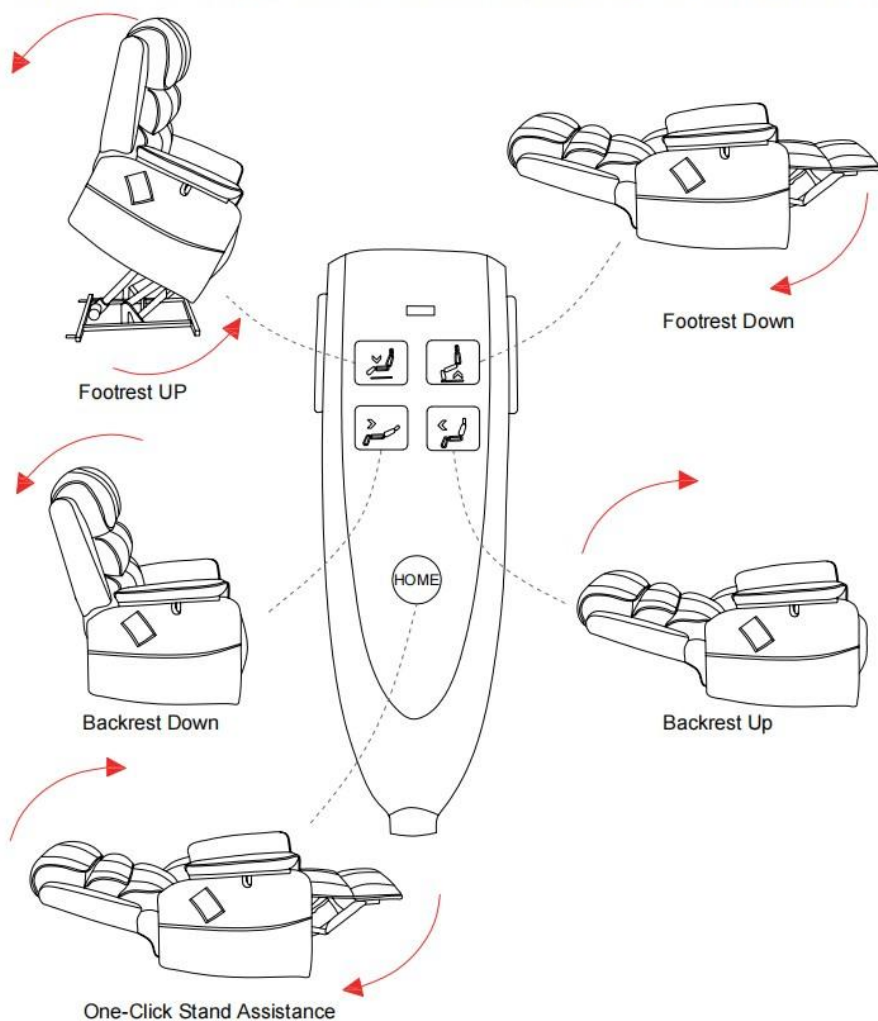Adjust the backrest and footrest independently to a full reclining position

TECHNICAL SPECIFICATIONS

Model	SX-81541-2
Rating(s)	DC29V 1.8A
Safe Loading Weight	350LB
Product Dimensions (LxWxH)	975*830*1075mm
Packing Size (LxWxH)	825*790*710mm
Net Weight (kg)	About 45kg
Gross Weight (kg)	About 50.3kg

*Products such as specifications, appearance, and design are subject to modification without prior notice.

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YH CONSULTING LIMITED.
C/O YH Consulting Limited Office 147, Centurion House, London Road, Staines-upon-Thames, Surrey, TW18 4AX

VEVOR[®]

TOUGH TOOLS, HALF PRICE

Technical Support and E-Warranty Certificate

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Importerad till AUS: SIHAO PTY LTD. 1 ROKEVA STREETEASTWOOD NSW 2122 Australien
Importerad till USA: Sanven Technology Ltd. Suite 250, 9166 Anaheim Place, Rancho Cucamonga, CA 91730
E-CrossStu GmbH