VEVOR[®]

TOUGH TOOLS, HALF PRICE

Technical Support and E-Warranty Certificate www.vevor.com/support

TOILET SAFETY RAILS

MODEL: FLB760599

We continue to be committed to provide you tools with competitive price.

"Save Half", "Half Price" or any other similar expressions used by us only represents an estimate of savings you might benefit from buying certain tools with us compared to the major top brands and does not necessarily mean to cover all categories of tools offered by us. You are kindly reminded to verify carefully when you are placing an order with us if you are actually saving half in comparison with the top major brands.

VEVOR®

TOUGH TOOLS, HALF PRICE

TOILET SAFETY RAILS

MODEL: FBL760599

NEED HELP? CONTACT US!

Have product questions? Need technical support? Please feel free to contact us:

Technical Support and E-Warranty Certificate
www.vevor.com/support

This is the original instruction, please read all manual instructions carefully before operating. VEVOR reserves a clear interpretation of our user manual. The appearance of the product shall be subject to the product you received. Please forgive us that we won't inform you again if there are any technology or software updates on our product.

Warning-To reduce the risk of injury, users must read the instructions manual carefully.

WARNING:

Read the manual completely to avoid harm, product damage, or warranty avoidance. Seek professional help if you encounter discomfort, pain or worsening symptoms. If necessary, use supervision in order to assemble and use this product correctly and safely. Do not exceed the 300 lbs weight limit.

Regularly check for secure assembly and contact support if in doubt. Ensure the product's legs are even and locked before use, and discontinue usage if there are signs of wear, damage, or missing parts. The product is to be used on flat, stable surfaces. Do not disassemble if not necessary, as it might compromise the longevity of the product. Do not modify the product unless specified in the manual. Use only parts approved by VEVOR for this specific product. Confirm the compatibility of the toilet rail with your toilet before first use. The product isn't built to withstand pressure or impact, and isn't to be used as a ladder or loaded with heavy objects.

Keep away from children and pets. Contains potentially hazardous small parts - choking hazard. Exercise caution during operation and assembly. Avoid extreme temperatures or open flames. If an allergic reaction occurs, cease use immediately and consult a healthcare professional.

Warranty Policy

We warrant your new VEVOR product to be free from defects in materials and workmanship 3 years of the original purchase. However, please keep in mind that our guarantee policy does not cover counterfeit or replica items, nor does it extend to products purchased from unauthorized resellers or in used condition.

Please note that each purchase is eligible for only one guarantee redemption, and we cannot repeatedly refund, replace, or exchange your product. We also kindly ask that you handle the product with care and use it according to the instructions provided. Unfortunately, we cannot cover damage resulting from misuse or abuse of the product. This warranty is not transferable and applies only to the original purchaser.

IMPORTANT

1. Don't exceed the weight capacity - 300 lbs max.
2. Use on stable surfaces only.
3. Regularly inspect for signs of wear, damage, or loose parts.
4. Clean regularly using a mild detergent and warm water, rinse off any soap residue. After cleaning, dry thoroughly.
5. Before use, confirm with your healthcare provider that the product meets your specific needs. The following guidelines are not a substitute for medical advice.

SAVE THESE INSTRUCTION

PARAMETER LIST

Model	FBL760599
Rated load (lbs)	300
Elevated height (in)	3.5
Net weight(kg)	2.01
Product size(mm)	440*610*275

PART LIST

When unpacking this product, check to make sure the parts listed below are included and carefully inspect for any damage that may have occurred during transit. If any part is missing or damaged, do not attempt to assemble or use the product.


NO	PICTURE	PART NUMBER	QTY	NO	PICTURE	PART NUMBER	QTY
1		Seat	1	6		Bolt	2
2		Handrail	2	7		Butterfly nut	2

3		Long L-shaped tube	2	8		Iron washer	2
4		Short L-shaped tube	2	9		Plastic washer	2
5		C-clip	4				

OPERATION


1. Installation Procedures

1.As shown in the figure, assemble the right handrail② by inserting the long ③and short L-shaped tubes ④into the handrail tubes in order of their perforated ends, and then inserting the two C-clips into the aligned holes from the opposite sides of the two L-shaped tubes in order of their perforated ends. (Note: the long L-tube is on the lower side of the right diagram)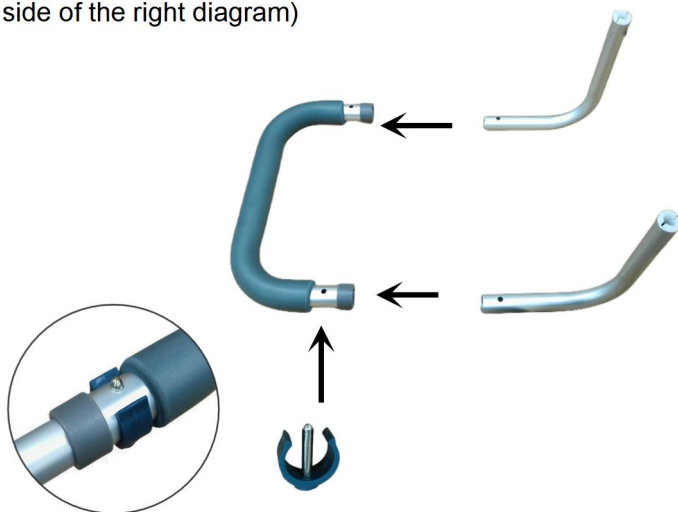


Insert the C-clip
as shown in the
right figure.

2. Follow the previous steps to install the left handrail.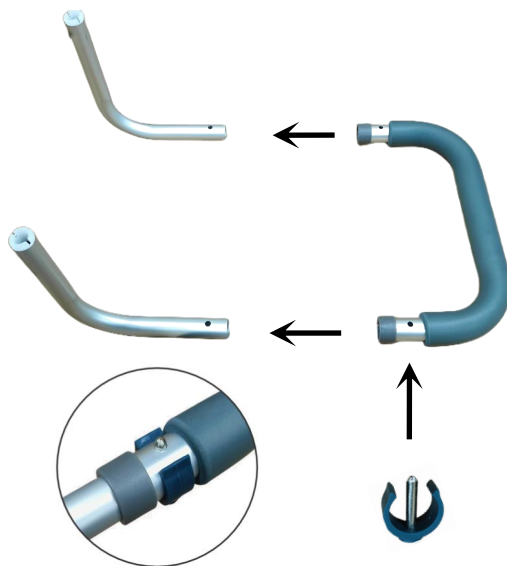

Insert the C-clip
as shown in the
right figure.

3. As shown in the figure, insert the assembled left/right handrail into the tube opening of the seat cushion ① in the direction of the figure below until the marbles of the long L-shaped tube ③ and the short L-shaped tube ④ are smoothly ejected from the holes of the short tube at the bottom of the seat. (Note: The long L-shaped tube is at the front of the seat cushion)

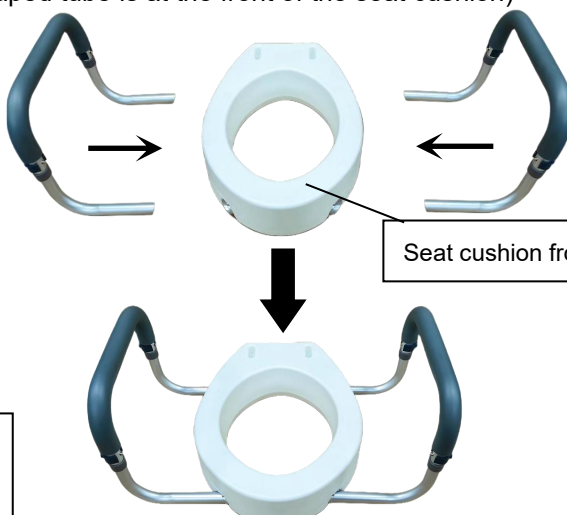Pictures of the
completed
assembly

4. As shown in the figure, cover the toilet's original seat plate and seat cover, locate the nut that secures the seat plate and seat cover, and remove the toilet's original seat plate and seat cover by turning the butterfly nut counterclockwise.

5. As shown in the figure, place the assembled seat on the toilet, then place the toilet's original seat plate and seat plate cover so that the fixing holes for the toilet, seat, seat plate and seat plate cover are aligned.

6. As shown in the figure, insert the bolt ⑥ into the fixing holes aligned with the toilet, seat, seat plate and seat plate cover, and then put the plastic washer, iron washer and butterfly nut on the other end of the bolt in turn and turn the butterfly nut clockwise to lock it.

Manufacturer: Zhongshan Bliss Medical Instrument Co., Ltd
Address: 3-6F & 7F-1, No. 63, Tongxing East Road, Yumin Community Xiaolan, Zhongshan, Guangdong, 528414, CHINA
Imported to AUS: SIHAO PTY LTD. 1 ROKEVA STREETEASTWOOD NSW 2122 Australia
Imported to USA: Sanven Technology Ltd. Suite 250, 9166 Anaheim Place, Rancho Cucamonga, CA 91730

UK	REP
-----------	------------

Pooledas Group Ltd
Unit 5 Albert Edward House, The Pavilions
Preston, United Kingdom

EC	REP
-----------	------------

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VEVOR[®]

TOUGH TOOLS, HALF PRICE

Technical Support and E-Warranty Certificate

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Tillverkare: Zhongshan Bliss Medical Instrument Co., Ltd. **Address:** 3-6F &
Importerad till AUS: SIHAO PTY LTD. 1 ROKEVA
Importerad till USA: Sanven Technology Ltd. Suite 250, 9166 Anaheim Place,
SHUNSHUN GmbH