VEVOR[®]

TOUGH TOOLS, HALF PRICE

Technical Support and E-Warranty Certificate www.vevor.com/support

PHYSICIAN SCALE

Model: SF-891 / SF-884

We continue to be committed to provide you tools with competitive price.


"Save Half", "Half Price" or any other similar expressions used by us only represents an estimate of savings you might benefit from buying certain tools with us compared to the major top brands and does not necessarily mean to cover all categories of tools offered by us. You are kindly reminded to verify carefully when you are placing an order with us if you are actually saving half in comparison with the top major brands.

VEVOR[®]

TOUGH TOOLS, HALF PRICE

PHYSICIAN SCALE

Model: SF-891、SF-884

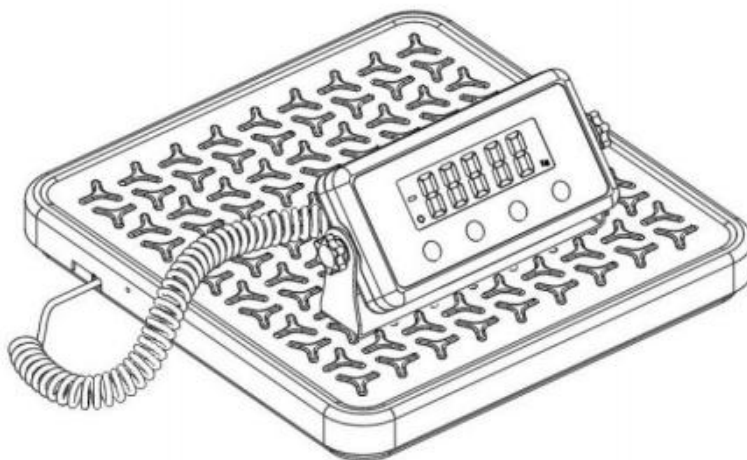NEED HELP? CONTACT US!

Have product questions? Need technical support? Please feel free to contact us:

Technical Support and E-Warranty Certificate
www.vevor.com/support

This is the original instruction, please read all manual instructions carefully before operating. VEVOR reserves a clear interpretation of our user manual. The appearance of the product shall be subject to the product you received. Please forgive us that we won't inform you again if there are any technology or software updates on our product.

SAFETY WARNINGS AND PRECAUTIONS

	Warning-To reduce the risk of injury, user must read instructions manual carefully.
	CORRECT DISPOSAL This product is subject to the provision of european Directive 2012/19/EU. The symbol showing a wheelie bin crossed through indicates that the product requires separate refuse collection in the European Union. This applies to the product and all accessories marked with this symbol. Products marked as such may not be discarded with normal domestic waste, but must be taken to a collection point for recycling electrical and electronic devices.
	Indoor Use Only
	Compliance is a EC security certification.
	FCC Information: CAUTION: Changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment! This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions: 1) This product may cause harmful interference. 2) This product must accept any interference received, including interference that may cause undesired operation. WARNING: Changes or modifications to this product not expressly approved by the party responsible for compliance could void the user's authority to operate the product. Note: This product has been tested and found to comply with the limits for a Class B digital device pursuant to Part 15 of the FCC Rules, These limits are designed to provide reasonable protection against harmful interference in a residential installation. This product generates, uses and can radiate radio frequency energy, and if not installed and used in accordance with the instructions, may cause harmful interference to radio

	communications. However, there is no guarantee that interference will not occur in a particular installation. If this product does cause harmful interference to radio or television reception, which can be determined by turning the product off and on, the user is encouraged to try to correct the interference by one or more of the following measures.
--	---

- Reorient or relocate the receiving antenna.
- Increase the distance between the product and receiver.
- Connect the product to an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for assistance.

Thank you for using this product. Please read all of the instructions before using it. The information will help you achieve the best possible results.

WARNING:

Please read this manual carefully before using the product. Failure to do so may result in serious injury.

ELECTRIC SAFETY!

1. Before using, check that the voltage power corresponds to the one shown on the appliance nameplate. Ensure that the voltage set on the machine corresponds to the electricity supply in your country.
2. Do not operate any appliance with a damaged cord or plug or after it malfunctions or has been damaged in any manner. Return the appliance to the nearest authorized service facility for examination, repair, or adjustment.
3. If the supply cord is damaged, it must be replaced by a special cord or assembly available from the manufacturer or its service agent.
4. Do not let the cord hang over the edge of a table or counter or touch hot surface.
5. Do not use appliances for other than intended use.
6. To protect against fire, electric shock, or personal injury, do not immerse cords, electric plugs, or appliances in water or other liquids.
7. The use of accessories, which the appliance manufacturer does not recommend, may result in fire, electric shock, or personal injury.
8. Do not use an extension cord with this product.

9. The power cord should lie flat on a table or counter surface and may not hang over the edge. Please keep it away from hot surfaces and water.
10. **WARNING:** This product contains small parts. If swallowed, it will pose a choking hazard. Keep these small parts away from children when assembling!

ATTENTION!

WARNING! Remove all packaging materials and keep them for future use. Packaging materials are not a toy.

WARNING! Don't let children play with the packaging materials. If they swallow the packaging materials, they will suffocate!

SAVE THESE INSTRUCTIONS

Thank you for selecting this digital scale, which provides clear and easily readable weight measurements. With its anti-slip platform, it ensures stability and safety, making it suitable for individuals with reduced mobility or those who may require additional support. This scale is ideal for a variety of settings including medical facilities, health clinics, gyms, weight management programs, schools, and for personal use at home

1. Technical Specifications

Capacity × Graduation	100kg*50g、100kg*100g、 200kg*50g、200kg*100g、 250kg*50g、250kg*100g
Weighing units	Kg,lb
Weighing pan size	12.9" x 12.5"
Overload indication	〔O_Ld〕
Low voltage indication	〔LO〕
Operating temperature	5℃-35℃
Power Supplier	4 x AA(1.5V) batteries or AC adapter

2. Introductions of Button

- (1) **ON/OFF BUTTON:** Press ON/OFF button to turn on/off the scale.

- (2) **UNIT BUTTON:** Press Mode button, you can switch units.
- (3) **HOLD BUTTON:** Lock/Unlock current weighing result.
- (4) **TARE BUTTON:** Enter/Exit tare mode, the tare function will eliminate the weight of the load.

3. Operation

- (1) Place the scale on the solid & flat surface.
- (2) Press ON/OFF button to turn on the scale, the scale starts a zeroing procedure which from 88.8.8.8 to 0.00kg/0.0LB.
- (3) After the scale shows zero, start your weighing.
- (4) Press Unit button, you can switch units between lb and kg.

Notice: Only after the screen show zero, you can start weighing.

When turning on the scale, there must be now eight on the scale.

While the battery or power supply is connected, the scale will automatically turn on.

4. About Automatically Shutting Down

Powered by batteries, if there is no operation, the machine will automatically shut down after 5 minutes.

Powered by electricity, the machine will not automatically shut down unless you press ON/OFF button.

5. Tips of Calibration (each scale has been calibrated before leaving the factory)

- (1) In an environment without strong magnetic field interference or strong convection, put the scale on the flat surface.
- (2) Turn on the scale, press and hold the“unit ”button for 5 seconds, the screen shows“CAL then you can only see “kg” the screen.
- (3) Press the “UNIT” button, choose calibration unit between kg and lb, then press TARE to confirm the unit.
- (4) Press unit to choose the calibration weight that you will put(10kg, 20kg, 50kg), then press TARE to confirm the weight.
- (5) The weight value that you choose in the previous step will flash on the screen, put counterweight on the scale, then press TARE, the screen will show“ PASS ” Congratulations, the calibration procedure is done.

Notice:

If you want to force quit the calibration procedure, press and hold ON/OFF button.

6. Tips of Calibration (each scale has been calibrated before leaving the factory)

- (1) In an environment without strong magnetic field interference or strong convection, put the scale on the flat surface.
- (2) Turn on the scale, press and hold the“unit ”button for 5 seconds, the screen shows“CAL then you can only see “kg” the screen.
- (3) Press the “UNIT” button, choose calibration unit between kg and lb, then press TARE to confirm the unit.
- (4) Press unit to choose the calibration weight that you will put(10kg, 20kg, 50kg), then press TARE to confirm the weight.
- (5) The weight value that you choose in the previous step will flash on the screen, put counterweight on the scale, then press TARE, the screen will show“ PASS ” Congratulations, the calibration procedure is done.

Notice:

If you want to force quit the calibration procedure, press and hold ON/OFF button.

7. Care Instructions

- (1) Take and put gently.
- (2) The machine should be used on the solid flat and stable surface, in an environment without shock, fluently , radiation, and strong electromagnetic interference. The four feet of the scale should be on the same level.
- (3) Please use within capacity(550lb), overload will permanently damage the sensor.
- (4) Don't leave weight on the scale when not in use.
- (5) Weighing Range 0.5lb - 550lb (0.25kg - 250kg).

Notice:

8. The batteries are non-rechargeable.

9. Automatic shutdown time setting

(1) Powered by batteries, if there is no operation, the machine will automatically shut down after 5 minutes.

(2) In the zero state, long-press “TARE” button to enter the shutdown time selection.

Press “TARE” button to switch between “1” (minutes) , “3” (minutes) , “5” (minutes) and “-” (No shutdown) .

Press the “MODE” button to confirm.

(3) Powered by electricity, the scale will not automatically shut down unless you press ON/OFF button.

10. If the customer enters the setting state by mistake, you can long press the ON/OFF button to turn off the scale, or remove the battery to exit the setting state.

11. Pay attention to the waterproofness, the product is not waterproof.

QUESTIONS	MEASURES
There is no response after pressing ON/OFF key	Check whether the power supply is normal Try another powering style, battery or adaptor
A negative number is displayed on the screen	Press TARE button to reset to zero
The number keeps beating and can't fix at zero	Check whether scale is on a solid flat surface and if there is an external force on the scale pan, such as artificial pressure, strong magnetic field interference or strong convection
There is no commodity on the scale, but the display is not zero	Check that whether scale is on a solid flat surface and whether there is strong electromagnetic interference in the environment. If not the above two factors, please press TARE button or calibrate the scale
Does not read the accurate weights	If trying several items and finding that the readings are inaccurate, please calibrate scale

Manufacturer: Shanghaimuxinmuyeyouxiangongsi
Address: Shuangchenglu 803nong11hao1602A-1609shi, baoshanqu, shanghai 200000 CN.
Imported to AUS: SIHAO PTY LTD.
1 ROKEVA STREETEASTWOOD NSW 2122 Australia
Imported to USA: Sanven Technology Ltd.
Suite 250, 9166 Anaheim Place, Rancho Cucamonga, CA 91730

EC	REP
----	-----

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YH CONSULTING LIMITED.
C/O YH Consulting Limited Office 147,
Centurion House, London Road,
Staines-upon-Thames, Surrey, TW18 4AX

VEVOR[®]

TOUGH TOOLS, HALF PRICE

Technical Support and E-Warranty Certificate
www.vevor.com/support

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Importerad till AUS: SIHAO PTY LTD.
Importerad till USA: Sanven Technology Ltd.
E-CrossStu GmbH