VEVOR[®]

TOUGH TOOLS, HALF PRICE

Technical Support and E-Warranty Certificate www.vevor.com/support

TOILET SAFETY RAILS

MODEL: FBL900633

We continue to be committed to provide you tools with competitive price.


"Save Half", "Half Price" or any other similar expressions used by us only represents an estimate of savings you might benefit from buying certain tools with us compared to the major top brands and does not necessarily mean to cover all categories of tools offered by us. You are kindly reminded to verify carefully when you are placing an order with us if you are actually saving half in comparison with the top major brands.

VEVOR®

TOUGH TOOLS, HALF PRICE

TOILET SAFETY RAILS

MODEL: FBL900633

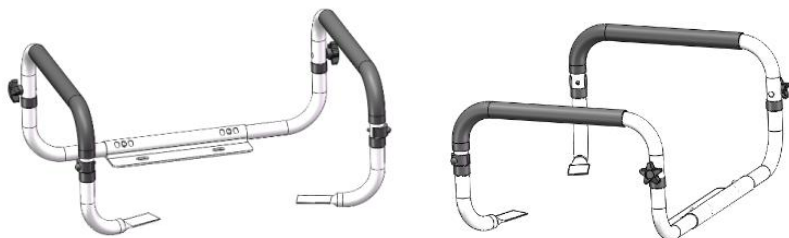NEED HELP? CONTACT US!

Have product questions? Need technical support? Please feel free to contact us:

Technical Support and E-Warranty Certificate
www.vevor.com/support

This is the original instruction, please read all manual instructions carefully before operating. VEVOR reserves a clear interpretation of our user manual. The appearance of the product shall be subject to the product you received. Please forgive us that we won't inform you again if there are any technology or software updates on our product.

Warning-To reduce the risk of injury, users must read the instructions manual carefully.

WARNING:

Read the manual completely to avoid harm, product damage, or warranty avoidance. Seek professional help if you encounter discomfort, pain, or worsening symptoms. If necessary, use supervision in order to assemble and use this product correctly and safely. Do not exceed the 300lb weight limit.

Regularly check for secure assembly and contact support if in doubt. Ensure the product's legs are even and locked before use, and discontinue usage if there are signs of wear, damage, or missing parts. The product is to be used on flat, stable surfaces. Do not disassemble if not necessary, as it might compromise the longevity of the product. Do not modify the product unless specified in the manual. Use only parts approved by PELEGON LLC for this specific product. Confirm the compatibility of the toilet rail with your toilet before first use. The product isn't built to withstand pressure or impact and isn't to be used as a ladder or loaded with heavy objects.

Keep away from children and pets. Contains potentially hazardous small parts - choking hazard. Exercise caution during operation and assembly. Avoid extreme temperatures or open flames. If an allergic reaction occurs, cease use immediately and consult a healthcare professional.

Warranty Policy

We warrant your new VEVOR product to be free from defects in materials and workmanship 3 years of the original purchase. However, please keep in mind that our guarantee policy does not cover counterfeit or replica items, nor does it extend to products purchased from unauthorized resellers or in used condition. Please note that each purchase is eligible for only one guarantee redemption, and we cannot repeatedly refund, replace, or exchange your product. We also kindly ask that you handle the product with care and use it according to the instructions provided. Unfortunately, we cannot cover damage resulting from misuse or abuse of the product. This warranty is not transferable and applies only to the original purchaser.

IMPORTANT

- 1. Don't exceed the weight capacity - 300 lb max.
- 2. Use on stable surfaces only.
- 3. Regularly inspect for signs of wear, damage, or loose parts.
- 4. Clean regularly using a mild detergent and warm water, rinse off any soap residue. After cleaning, dry thoroughly.
- 5. Before use, confirm with your healthcare provider that the product meets your specific needs. The following guidelines are not a substitute for medical advice.

SAVE THESE INSTRUCTION


PARAMETER LIST

Model	FBL900633
Rated bearing capacity (lbs)	300
Width adjustment range (mm)	410-510
Net weight	1.83kg
Product size (mm)	475-575*425*263

PART LIST



When unpacking this product, check to make sure the parts listed below are included and carefully inspect for any damage that may have occurred during transit. If any part is missing or damaged, do not attempt to assemble or use the product.





NO	PICTURE	PART NUMBER	QTY	NO	PICTURE	PART NUMBER	QTY
1		Flattened tube	2	6		Handrail tube right	1
2		Figure 6 regulating tube	1	7		Handrail tube left	1
3		Proximity Card	2	8		L-shaped tube right	1
4		rotary knob	2	9		L-shaped tube left	1
5		outer hexagon screw	2				


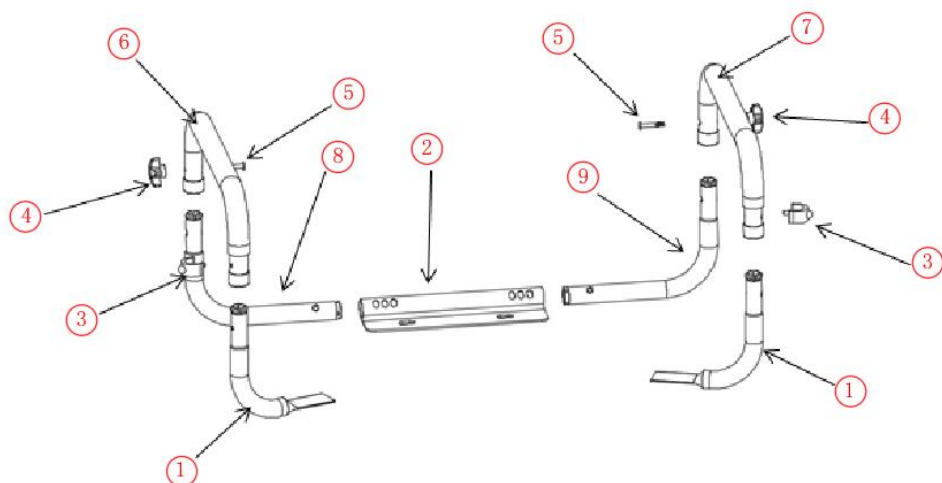
OPERATION

1. Installation Procedures

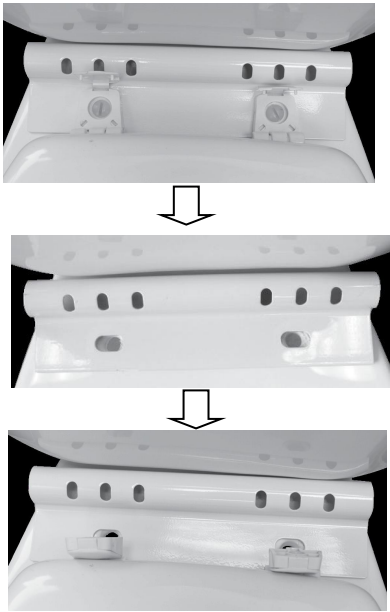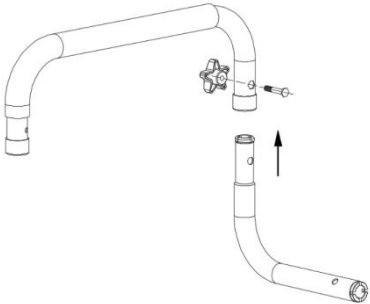	1. Align the bolt holes on the 6-figure tube, and toilet seat, and secure the other end of the bolt.
	2. Install the L-shaped tube left to the top left of the handrail tube (1 5-star knob, 1 outer hex screw)

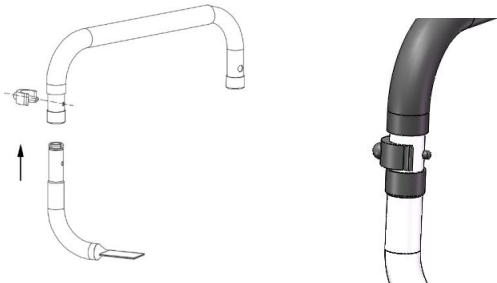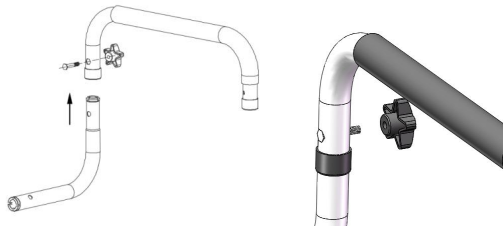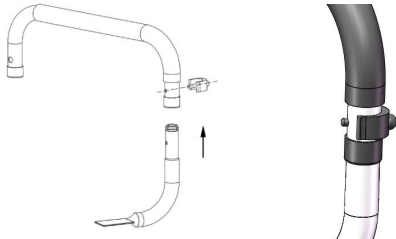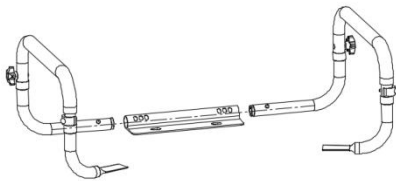	3. Install the flat tube on the upper left side of the handrail tube (1 type C card)
	4. Install the L-shaped tube right on the top right of the handrail tube (1 5-star knob, 1 outer hex screw)
	5. Install the flat tube on the top right of the handrail tube (1 type C card)
	6. Install the assembled left handrail tube and right handrail tube on the adjustment tube (figure-6), and the assembled product as shown in the following figure

In the picture below, we insert the mounting hole into the pin of the toilet.

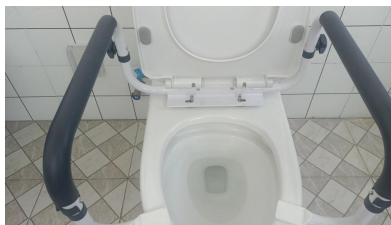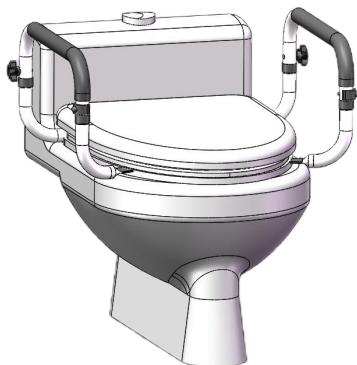2. Installation Procedures

- With the tailgate open, place Bed Extender pivot pins into the left and right mounting brackets. Rotate to 45 until it drops into place, then lower it onto the open tailgate.
- Flip the Bed Extender out to contain cargo in the truck bed when the tailgate is open. **CAUTION:** 200lb maximum load on the open tailgate. Always distribute the cargo evenly and properly secure inside the bed. Do not secure cargo to the Bed Extender or the tailgate.
- Before driving, secure Bed Extender by locking the provided strap buckles into the truck's tailgate latches. To release, pull the tailgate handle. **NOTE:** Ensure that both buckles are properly locked into the latches by pulling up on each strap.
- Flip the Bed Extender forward and close the tailgate to keep smaller cargo safe and organized.

Address: Baoshanqu Shuangchenglu 803long 11hao 1602A-1609shi
Shanghai

Imported to AUS: SIHAO PTY LTD. 1 ROKEVA STREETEASTWOOD
NSW 2122 Australia

Imported to USA: Sanven Technology Ltd. Suite 250, 9166 Anaheim
Place, Rancho Cucamonga, CA 91730

Pooledas Group Ltd
Unit 5 Albert Edward House, The Pavilions
Preston, United Kingdom

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VEVOR[®]

TOUGH TOOLS, HALF PRICE

Technical Support and E-Warranty Certificate

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Importerad till AUS: SIHAO PTY LTD. 1 ROKEVA STREETEASTWOOD
Importerad till USA: Sanven Technology Ltd. Suite 250, 9166
SHUNSHUN GmbH