VEVOR[®]

TOUGH TOOLS, HALF PRICE

Technical Support and E-Warranty Certificate

www.vevor.com/support

First aid kits User Manual

MODEL:RS018/RS037/RS035/RH076

We continue to be committed to provide you tools with competitive price. "Save Half", "Half Price" or any other similar expressions used by us only represents an estimate of savings you might benefit from buying certain tools with us compared to the major top brands and does not necessarily mean to cover all categories of tools offered by us. You are kindly reminded to verify carefully when you are placing an order with us if you are actually Saving Half in comparison with the top major brands.

VEVOR®

TOUGH TOOLS, HALF PRICE

First aid kits

MODEL:RS018/RS037/RS035/RH076

(The picture is for reference only, please refer to the actual object)

NEED HELP? CONTACT US!

Have product questions? Need technical support? Please feel free to contact us:

Technical Support and E-Warranty Certificate
www.vevor.com/support

This is the original instruction, please read all manual instructions carefully before operating. VEVOR reserves a clear interpretation of our user manual. The appearance of the product shall be subject to the product you received. Please forgive us that we won't inform you again if there are any technology or software updates on our product.

	Warning-To reduce the risk of injury, user must read instructions manual carefully.
	This product is subject to the provision of European Directive 2012/19/EC. The symbol showing a wheeled bin crossed through indicates that the product requires separate refuse collection in the European Union. This applies to the product and all accessories marked with this symbol. Products marked as such may not be discarded with normal domestic waste, but must be taken to a collection point for recycling electrical and electronic devices

BLEEDING

Severe bleeding is a major threat to life and always needs prompt attention by the First Aider.

1. Apply direct pressure to the wound using your hand or the casualty's hand

X: 9.92 cm

2. Elevate the v: 5.11 cm

3. Apply a thick dressing and a firm bandage.

4. Check circulation in the end of the bandage limb

5. Get medical help.

EMBEDDED OBJECTS

1. Apply pressure with pads around the embedded object.

2. Apply a firm bandage to hold the pads in place,

3. Get medical help.

EYE INJURIES

LARGE EMBEDDED FORCING BODY OR LACERATION

1. Assist the casualty into a comfortable position
2. Tell the casualty not to rub or move the eyes.
3. Cover both eyes with a light dressing and bandage.
4. Get medical help

SMALL FOREIGN BODY IN THE EYE

1. Assist the casualty into a sitting position with head back and leaning to one side
2. Gently separate the eyelids.
3. Irrigate the eye with clean water or a sterile saline solution.
4. Get medical help.
5. Do Not use any object to wipe the eye.

SPINAL INJURIES

UNCONSCIOUS CASUALTY

1. Gently roll the casualty onto their side with no twisting and minimal movement
2. Support the head and neck in alignment with the rest of the body.
3. Maintain support of the head and neck with both of your hands placed either side of the casualty's head until help arrives.
4. Get medical help.

IMPORTANT the management principles of DRABC ALWAYS take priority in the care and management of a suspected spinal injury.

CONSCIOUS CASUALTY

1. Tell the casualty to remain still
2. Immobilize the casualty's head by placing hands on both sides
3. Loosen any constrictive clothing.
4. Calmly reassure the casualty
5. Get medical help.

DO NOT move casualty unless danger is life-threatening

BITES & STINGS

FUNNELWEB SPIDER & SNAKES

1. Reassure the casualty.
2. Assist the casualty into a comfortable position; usually lying on the back.
3. Apply a pressure immobilization bandage over the entire limb.
- d. Get medical help

REDBACK SPIDER

1. Rest and reassure the casualty.
2. Apply a cold compress (wrapped in a towel/cloth) to bite area.
3. Get medical help.

BEESTING

1. Rest and reassure the casualty
2. Scrap off sting sideways with finger-nail or sharp edge
3. Apply a cold compress (wrapped in a towel/cloth to bite area.
4. Get medical help.

BURNS & SCALDS

The immediate aim of treatment is to COOL the burned area.

1. Cool the burn under cold running water for a minimum of 15 minutes.
2. Remove items of clothing/jewellery likely to retain heat.
3. Cover with a loose sterile/clean dressing, e.g. hankerchief: pillowcase
4. Elevate burnt limbs
5. Get medical help,

CHEST PAINS

1. Assist the casualty into a comfortable position
2. Ask the casualty if he/she has experienced this before
3. Assist the casualty to take their prescribed medication for chest pains

IMPORTANT

Chest pains can conceal fatal conditions,
There must be no delay in seeking medical advice.

DRABS-Action Plan

DANGER.RESPONSE.AIRWAY.BREATHING.CIRCULATION

1. DANGER

The first step in DRABC is to look for, assess and where possible control any dangers that can injure:

- 1.The first aider
- 2.The bystanders
- 3.The casualty

These dangers may include fire, moving traffic,poisonous gases,electricity sharp objects etc.

CALL FOR HELP

When calling or sending a bystander for help, the following details must be given to the emergency services.

- 1.The exact location-house number, street name,nearest cross street and or landmark,town and even state
- 2.What has happened
- 3.How many people are hurt
- 4.If anyone is trapped
- 5.DO NOT hang-up before the operator has finished. Let operator hang-up first

6.If sending a bystander for help tell him come back to me after you have made the call.This confirms that the call have been made.

2.RESPONSE


The second step in DRABC is used to determine whether a casualty is conscious or unconscious.

The Response is easily checked by a Gentle Shake and Shout. This is done by

- 1.Gently shaking the casualty's shoulders
- 2.Calling out in a loud voice "Can you hear me? Open you eyes.
- 3.The casualty's hands can also be squeezed to get a response,

IF THERE IS NO RESPONSE

gently roll the casualty onto the side while supporting the head.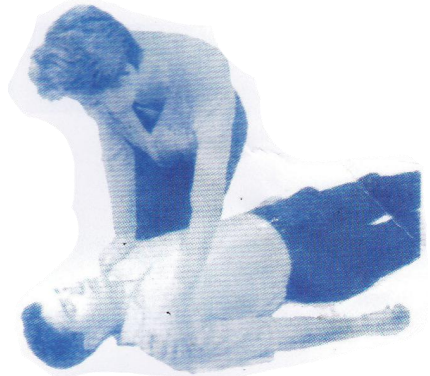


3.AIRWAY

The third step in DRABC is to check and ensure the casualty's airway is clear of obstructions.

- 1.Turn the casualty on the side
- 2.Turn face slightly down and tilt head back
- 3.Open mouth and look inside for any obstructions.
- 4.Sweep out any obstructions with your fingers.

By tilting the head back the tongue is also moved away from the back of the throat and helps clear the airway.

If the casualty's check is clear or has been cleared proceed to the next step of DRABC
Check the Breathing

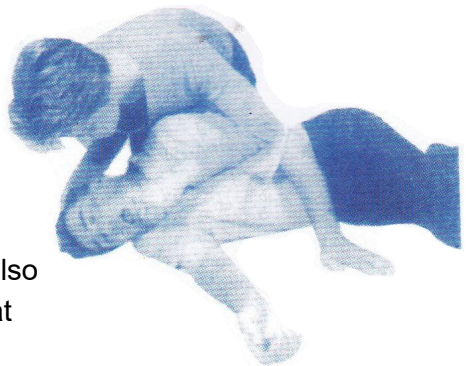4. BREATHING

The fourth step in DRABc is to check whether the casualty is breathing by 'Look, Listen and Feel'.

1. Look for movement of the casualty's chest and upper abdominal area.
2. Listen for air movement out of the casualty's mouth and nose
3. Feel for air movement against the side of your face


IF BREATHING IS ABSENT

Immediately start EAR (Expired Air Resuscitation), usually Known as mouth-to-mouth resuscitation. Mouth-to-nose resuscitation may also be used.

EAR


1. Roll casualty onto the back
2. Tilt head backwards
3. Seal casualty's mouth
4. Give up to 5 full breaths in 10 seconds

NOTE: For the first aider's protection the use of a shield device is recommended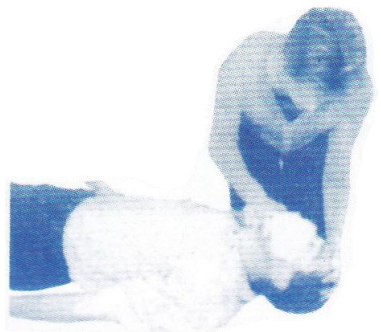

RECOVERY POSITION

When breathing begins place victim on the side, head backjaw supported face pointing slightly towards the ground

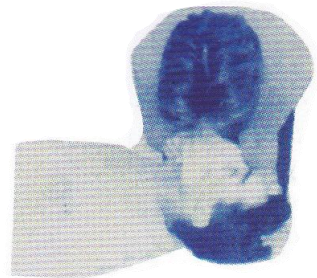5.CIRCULATION

The last step in DRABC is to check whether the casualty has a pulse.

The best place to check for a pulse is on either side of the neck

1. Place two or three fingers over the casualty's Adam's Apple.
- 2.Allow the fingers to slide off the Adam's Apple into the curve between the Adam's Apple and the neck muscle
- 3.Start CPR if the casualty has no pulse

CPR

- 1.Place heel of hand on lower half of the breastbone
- 2.Compress chest 4-5cm 15 times and give 2 breaths
- 3.Repeat 4 times per minute

Expired Air Resuscitation & Cardio Pulmonary Rate Chart

Adult& Young Child Infant Older Child			
Head Tilt	Maximum	Maximum	Nil
Breaths	Full	Small	puffs
BAR Rate	1:4Sec	1:3Sec	1:3Sec
Breaths/Min	15	20	20
CPR Pressure	2 Hands	1 Hands	2 Fingers
CPR Depth	4-5cm	1/3 of Chest	1/3 of Chest
CPR Rate	80-100/min	100/min	100/min
Ratio of Compressions to Inflation			
1 Rescuer	15:2	5:1	5:1
Cycles Per Min	4	12	12
2 Rescuer	5:1	5:1	5:1
Cycles Per Min	12	12	12

Important Note!

These first aid notes are not a substitute for proper first aid training. We recommend participating in formal first aid training as provided by an accredited first aid training provider.

Manufacturer: NINGBO TIANBO FIRST AID PRODUCT CO., LTD.

Address: Yushantou village,Dongqiao Town,Haishu, Ningbo, Zhejiang, 315000, CHINA. .

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YH CONSULTING LIMITED.
C/O YH Consulting Limited Office 147, Centurion House,
London Road, Staines-upon-Thames, Surrey, TW18 4AX

Technical Support and E-Warranty Certificate
www.vevor.com/support

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Tillverkare: NINGBO TIANBO FIRST AID PRODUCT CO., LTD.
Importerad till AUS: SIHAO PTY LTD, 1 ROKEVA STREETEASTWOOD NSW 2122 Australien
Importerad till USA: Sanven Technology Ltd., Suite 250, 9166 Anaheim Place, Rancho Cucamonga, CA 91730
E-CrossStu GmbH